

- ENVIRONMENTAL PARAMETERS -

OPERATING CONDITIONS



SHIPPING & STORAGE CONDITIONS



Shipping: The materials and components used in the construction of this device were selected to ensure that the device could be shipped by any standard commercial method without special handling conditions.

- DUTY CYCLE -

The MicroAire EpiCut is intended for single-use only. It should not be resterilized. For some procedures more than one EpiCut may be required.

- WARRANTY -

This product is warranted to be free from defects in manufacture, and this warranty is limited to the replacement of the product at no charge. This warranty is void in the event of product misuse, alteration or if the "Use By" date (📅) has passed.



Instructions For Use MicroAire[®] EpiCut[™] REF PL-400 and EpiCut[™] 55 REF PL-455



- INDICATIONS FOR USE -

The MicroAire EpiCut is designed to remove strips of epithelial tissue. The EpiCut is intended to be a replacement for scissors or scalpels when performing de-epithelialization.

MicroAire Surgical Instruments LLC

3590 Grand Forks Boulevard
Charlottesville, VA 22911 U.S.A.
(800) 722-0822 (434) 975-8000
Order Fax (800) 648-4309 or (434) 975-4131
www.microaire.com



MediMark Europe

11, rue Emile Zola - BP 2332
F-38033 Grenoble Cedex 2
France

-APPLICABLE PART NUMBERS-

REF Number	Description
PL-400	EpiCut
PL-455	EpiCut 55

- SYMBOL DEFINITIONS -



Caution, refer to accompanying documents.



European Conformity Mark with
MicroAire Notified Body Number



Single patient use only.
DO NOT RESTERILIZE



Date of Manufacture, YYYY-MM



Use by date, YYYY-MM.



Contents Sterile unless package is damaged or opened
or product has exceeded its "Use By" date.



Sterilization method, Gamma sterilization.



Product Catalog Number



Product Lot Number



Temperature Limitations



Humidity Limitations



Atmospheric Pressure Limitations



Authorized European Representative



Caution: Federal Law (U.S.A.) restricts this device
to sale by or on the order of a physician (or properly
licensed practitioner)

- INTRODUCTION -

Throughout the manual, the following terms are used to identify tips and precautions that will help avoid accidental injury to patients or personnel, or prevent damage to the system.

WARNING: Used to indicate that the safety of the patient and hospital personnel could be involved.

CAUTION: Used to point out special procedures or precautions that must be followed to avoid damaging the system/instrument.

NOTE: Used to point out the easiest means of carrying out techniques.

- GENERAL WARNINGS -

CAUTION: Federal Law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

CAUTION: The MicroAire EpiCut is a single use only product, and should be disposed of properly after use.

- EPICUT INSTRUCTIONS FOR USE -

1. Remove the EpiCut from the sterile packaging.
2. Remove the blade protector from the end of the EpiCut.
3. Use the EpiCut with the MicroAire EpiCut logo facing up.
4. Continue in this manner until the desired amount of epithelial tissue has been removed.