EN: INSTRUCTIONS FOR USE • (REF 7500-6XX) ASEPTIC BATTERY SYSTEM
DA: BRUGSANVISNING • (REF 7500-6XX) ASEPTISK BATTERISYSTEM
DE: GEBRAUCHSANWEISUNG • (REF 7500-6XX) ASEPTISCHES AKKUSYSTEM
ES: INSTRUCCIONES DE USO • SISTEMA DE BATERÍAS ASEPTICO (REF. 7500-6XX)
FR: INSTRUCTIONS D'UTILISATION • SYSTEME DE BATTERIE ASEPTIQUE (REF. 7500-6XX)
IT: ISTRUZIONI PER L'USO • SISTEMA DI BATTERIA ASEPTICA (REF 7500-6XX)
NL: GEBRUIKSAANWIJZING • (REF 7500-6XX) ASEPTISCH ACCUSYSTEEM
PT: INSTRUÇÕES PARA USO • (REF 7500-6XX) SISTEMA DE BATERIA ASEPTICO
FI: KÄYTTÖOHJEET • (REF 7500-6XX) ASEPTINEN AKKUAJAJESTELMÄ
SE: BRUKSANVISNING • (REF 7500-6XX) ASEPTISKT BATTERISYSTEM

CONTACT INFORMATION
MicroAire Surgical Instruments LLC, 3590 Grand Forks Boulevard, Charlottesville, VA 22911, USA
Customer Service, +1-(800) 722-0822, or +1-(434) 975-8000
Order Fax +1-(800) 648-4309 or +1-(434) 975-4131

EC REP MediMark Europe, 11, rue Emile Zola BP2332, F-38033 Grenoble Cedex 2, France
Must be collected separately from household waste. Dispose of as per WEEE Directive 2002/96/EEC

Waste Electrical and Electronic Equipment (WEEE) European Community Symbol. Batteries contain materials that must be recycled or disposed of properly. Do not dispose of as unsorted municipal waste. ALWAYS follow the current local recommendations and/or regulations governing environmental protection and the risks associated with recycling or disposing of the equipment at the end of its usual life. ALWAYS decontaminate a battery pack exposed to infectious material before sending it to a waste treatment facility.

REPAIR SERVICE
If a problem with your equipment should arise, contact our Customer Service Department at:
USA Telephone +1-800-722-0822 Fax +1-800-438-6309 Email inquiry@microaire.com
Outside of USA +1-434-975-8000 +1-434-975-4134 intlsv@microaire.com

©2017 MicroAire Surgical Instruments LLC
IM-ASEPBATT REV B Printed in USA

Table with 10 columns: SYMBOL, ENGLISH, DANSK, DEUTSCH, ESPAÑOL, FRANÇAIS, ITALIANO, NEDERLANDS, PORTUGUES, SUOMI, SVENSK. It provides translations for various symbols and terms used in the instructions.

EN: REF 7500-615 BATTERY HOUSING
DA: REF 7500-615 BATTERIHUS
DE: REF 7500-615 AKKUHÄUSE
ES: CARCASA DE BATERIA REF. 7500-615
FR : COMPARTIMENT DE LA BATTERIE REF. 7500-615
IT: RIF 7500-615 ALLOGGIAMENTO BATTERIA
NL: REF 7500-615 BEHUZING ACCU
PT: CARCAÇA DE BATERIA REF 7500-615
FI: REF 7500-615 AKKUKOTILO
SE: REF 7500-615 BATTERIHÖLJE

EN: REF 7500-625 CHARGER ADAPTER
DA: REF 7500-625 OPLADERADAPTER
DE: REF 7500-625 LADGERÄTADAPTER
ES: ADAPTADOR DEL CARGADOR REF. 7500-625
FR: ADAPTEUR DE CHARGEUR REF. 7500-625
IT: RIF 7500-625 ADATTORE CARICABATTERIE
NL: REF 7500-625 LADERADAPTER
PT: CARREGADOR DE BATERIA REF 7500-625
FI: REF 7500-625 LATAUSLAITTEEN SOVITIN
SE: REF 7500-625 LADDNINGSDAPTER

ENGLISH
INDICATIONS
The MicroAire Aseptic Battery System is intended for use with MicroAire U.L. Classified Battery Powered Instruments and Chargers.
COMPATIBLE DEVICES
Product Catalog Number Description
REF 6643/REF 6642B\* Battery SmartDriver Duo
REF 7505/REF 7500\* Drill/Reamer
REF 7506/REF 7600\* Osculating Saw
REF 7507/REF 7700\* Reciprocating Saw
REF 7800 Sternum Saw
REF 7500-700 Battery Charger

DANSK
INDIKATORER
MicroAire aseptisk batterisystem er beregnet til brug sammen med MicroAire U.L. klassificerede, batteridrevne instrumenter og opladere.
KOMPATIBLE ANORDNINGER
Produktets katalognummer Beskrivelse
REF 6643/REF 6642B\* Batteri SmartDriver Duo
REF 7505/REF 7500\* Bør/børstest
REF 7506/REF 7600\* Rundsag
REF 7507/REF 7700\* Bajonettsag
REF 7800 Sternumsag
REF 7500-700 Batterioplader

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**WARNING:** When closing the lid to the housing, keep fingers clear of it to avoid pinching fingers or gloves.

**Battery Attachment CAUTION:** Please consult your instrument manual for special concerns before connecting this battery pack.

To install the battery in an instrument, orient the battery with the knob towards the front of the instrument. Firmly push the battery into the base of the instrument until the battery latch clicks into place.

**WARNING:**  
 • The battery is fully seated and locked.  
 • Risk of fire. Replace Battery Pack only with a MicroAire 7500-620 Battery Pack.  
 • Explosion hazard. Not suitable for use in the presence of any flammable anesthetics or oxygen.  
 • Do not use in the presence of open flame or ignition sources.  
 • Use MicroAire 7500-700 Charger and 7500-625 Charger Adapter Kit to charge 7500-620 Battery Pack.

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