EN: INSTRUCTIONS FOR USE • (REF 7500-6XX) ASEPTIC BATTERY SYSTEM
DA: BRUGSANVISNING • (REF 7500-6XX) ASEPTISK BATTERISYSTEM
DE: GEBRAUCHSANWEISUNG • (REF 7500-6XX) ASEPTISCHES AKKUSYSTEM
ES: INSTRUCCIONES DE USO • SISTEMA DE BATERÍAS ASEPTICO (REF. 7500-6XX)
FR: INSTRUCTIONS D'UTILISATION • SYSTEME DE BATTERIE ASEPTIQUE (REF. 7500-6XX)
IT: ISTRUZIONI PER L'USO • SISTEMA DI BATTERIA ASEPTICA (REF 7500-6XX)
NL: GEBRUIKSAANWIJZING • (REF 7500-6XX) ASEPTISCH ACCUSYSTEEM
PT: INSTRUÇÕES PARA USO • (REF 7500-6XX) SISTEMA DE BATERIA ASEPTICO
FI: KÄYTTÖOHJEET • (REF 7500-6XX) ASEPTINEN AKKUUJÄRJESTELMÄ
SE: BRUKSANVISNING • (REF 7500-6XX) ASEPTISKT BATTERISYSTEM

CONTACT INFORMATION
MicroAire Surgical Instruments LLC, 3590 Grand Forks Boulevard, Charlottesville, VA 22911, USA
Customer Service, +1-(800) 722-0822, or +1-(434) 975-8000
Order Fax +1-(800) 648-4309 or +1-(434) 975-4131

EC REP MediMark Europe, 11, rue Emile Zola BP2332, F-38033 Grenoble Cedex 2, France
Must be collected separately from household waste.
Dispose of as per WEEE Directive 2002/96/EEC

Waste Electrical and Electronic Equipment (WEEE) European Community Symbol. Batteries contain materials that must be recycled or disposed of properly. Do not dispose of as unsorted municipal waste. ALWAYS follow the current local recommendations and/or regulations governing environmental protection and the risks associated with recycling or disposing of the equipment at the end of its usual life. ALWAYS decontaminate a battery pack exposed to infectious material before sending it to a waste treatment facility.

REPAIR SERVICE
If a problem with your equipment should arise, contact our Customer Service Department at:
USA Telephone +1-800-722-0822 Fax +1-800-438-6309 Email inquiry@microaire.com
Outside of USA +1-434-975-8000 +1-434-975-4134 intlsvc@microaire.com

©2021 MicroAire Surgical Instruments LLC
IM-ASEPBATT REV C Printed in USA 05/2021 1 of 2

Table with 10 columns: SYMBOL, ENGLISH, DANSK, DEUTSCH, ESPAÑOL, FRANÇAIS, ITALIANO, NEDERLANDS, PORTUGUES, SUOMI, SVENSK. It contains various symbols and their corresponding text in multiple languages.

EN: REF 7500-615 BATTERY HOUSING
DA: REF 7500-615 BATTERIHUS
DE: REF 7500-615 AKKUHÄUSE
ES: CARCASA DE BATERÍA REF. 7500-615
FR : COMPARTIMENT DE LA BATTERIE REF. 7500-615
IT: RIF 7500-615 ALLOGGIAMENTO BATTERIA
NL: REF 7500-615 BEHULZING ACCU
PT: CARCAÇA DE BATERIA REF 7500-615
FI: REF 7500-615 AKKUKOTILO
SE: REF 7500-615 BATTERIHÖLJE

EN: REF 7500-620 BATTERY PACK
DA: REF 7500-620 BATTERIPAKKE
DE: REF 7500-620 AKKUPACK
ES: PAQUETE DE BATERÍAS REF. 7500-620
FR : BLOC BATTERIE REF. 7500-620
IT: RIF 7500-620 GRUPPO BATTERIA
NL: REF 7500-620 ACCUSYSTEEM
PT: BATERIA REF 7500-620
FI: REF 7500-620 AKKUYKSIKÖ
SE: REF 7500-620 BATTERI PAKET

ENGLISH INDICATIONS
The MicroAire Aseptic Battery System is intended for use with MicroAire U.L. Classified Battery Powered Instruments and Chargers.
COMPATIBLE DEVICES
ENVIRONMENTAL CONDITIONS
WARRANTY
MicroAire Surgical Instruments warrants the 7500-ASEPBATT Aseptic battery system to be free from defects in material and workmanship in their manufacture for a period of 90 days from the original purchase date by the end customer.

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