

MicroAire LipoTower In-Service Checklist

(REF ASP-1021. See IM-LIPOTOWER for complete instructions.)

- The Main Power ON|OFF is located in the lower left corner on the back of the cart.



- The Cart Standby Power ON|OFF is located on the right side of the front bezel.



- The footswitch for the suction pump connects in the lower left corner on the back of the cart.



- The footswitch for the infiltration pump connects next to the roller pump on the side of the cart.



- Adjust the IV Pole by lifting the locking collar to raise or lower the pole.



- Waste collection canisters fit into the cup holder rings on the top of the cart.



- Insert infiltration tubing by lifting the latch on the roller pump on the side of the cart.



- The roller pump ON|OFF is located next to the roller pump on the side of the cart.



- The roller pump speed knob is below the switch.




- Vacuum Pump ON|OFF control can be selected by turning the three-way selector knob on the left side of the front bezel.




- Vacuum pressure can be adjusted by turning the MIN|MAX knob on the right side of the front bezel.



- Vacuum pressure is indicated on the LCD screen.

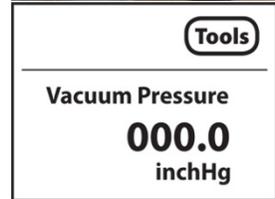

- The biofilter is located near the cup holder rings on the top of the cart.

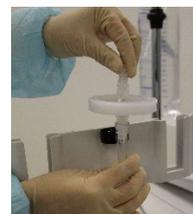

- The biofilter should be replaced after 50 hours of use, or sooner if performance is reduced.



- Reset the "Biofilter Hours" by selecting "Tools" from the main LCD screen, then selecting "Reset Biofilter to Zero."

