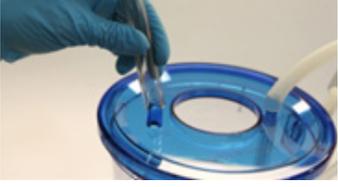


MicroAire LipoFilter® In-Service Checklist

SETUP INSTRUCTIONS

(REF ASP-CAN-2S. See IM-ASP-CAN-2S for complete instructions.)

<input type="checkbox"/> Place stand onto sterile field and secure with clamp.		<input type="checkbox"/> Connect fluid evacuation tubing to Y-connector tubing and the other end to a waste canister.	
<input type="checkbox"/> Attach fat evacuation tubing to bottom of canister and attach five-way connector to fat evacuation tubing.		<input type="checkbox"/> Secure fluid evacuation tubing in pigtail hook.	
<input type="checkbox"/> Place canister into stand.		<input type="checkbox"/> Connect liposuction tubing to "LIPO" port and the other end to the cannula/handpiece.	
<input type="checkbox"/> Attach plain side of Y-connector tubing to "EVAC" port.		<input type="checkbox"/> Pre-load canister with 500ml of sterile saline or Lactated Ringer's solution, then secure center cap.	
<input type="checkbox"/> Attach clamp side of Y-connector tubing to "CLAMP" port.			

MicroAire LipoFilter® In-Service Checklist

OPERATION INSTRUCTIONS

(REF ASP-CAN-2S. See IM-ASP-CAN-2S for complete instructions.)

<p><input type="checkbox"/></p> <p>With aspirator on, manually crimp liposuction tubing to prevent airflow and set vacuum level to approximately 457mmHg (18inHg).</p>		<p><input type="checkbox"/></p> <p>To extract fat, remove tubing from "LIPO" port and leave port open. Connect Toomey syringe to fat evacuation tubing.</p>	
<p><input type="checkbox"/></p> <p>Begin harvesting. Do not fill canister past 2500ml (marked "STOP").</p>		<p><input type="checkbox"/></p> <p>Open pinch-clamp on fat evacuation tubing and draw out remaining fluid. Close pinch-clamp and discard fluid.</p>	
<p><input type="checkbox"/></p> <p>Allow fat to separate from fluids (about 12 minutes).</p>		<p><input type="checkbox"/></p> <p>Reattach Toomey syringe, open pinch-clamp, draw out fat, close pinch-clamp, and remove syringe. Repeat to extract more fat.</p>	
<p><input type="checkbox"/></p> <p>To transfer fluid to waste canister, close pinch-clamp on Y-connector tubing. To stop fluid evacuation, open pinch-clamp. Continue harvesting if needed.</p>		<p>For additional harvesting, reload canister with 500ml of sterile saline or Lactated Ringer's solution and repeat.</p>	

