




PAL SYNERGY

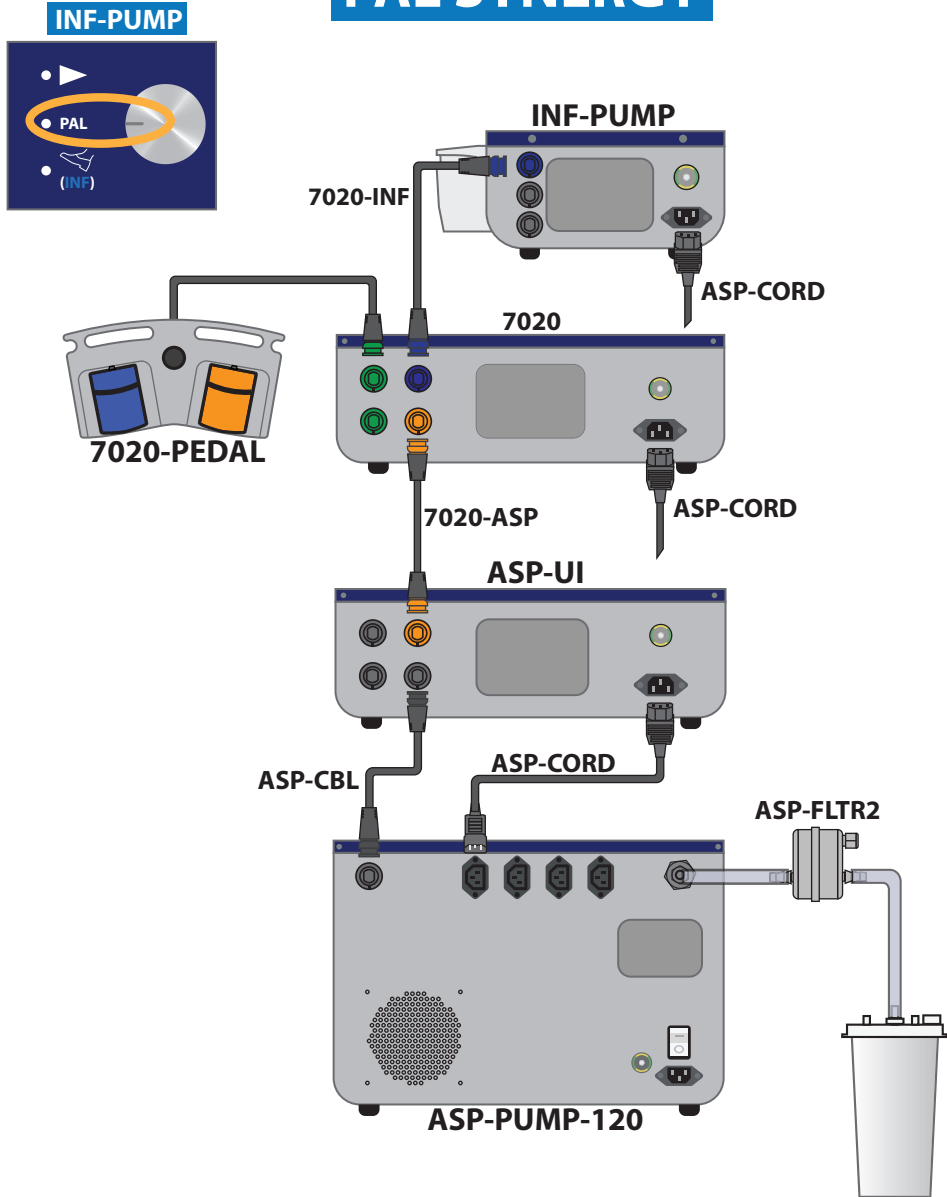

ASP ONLY

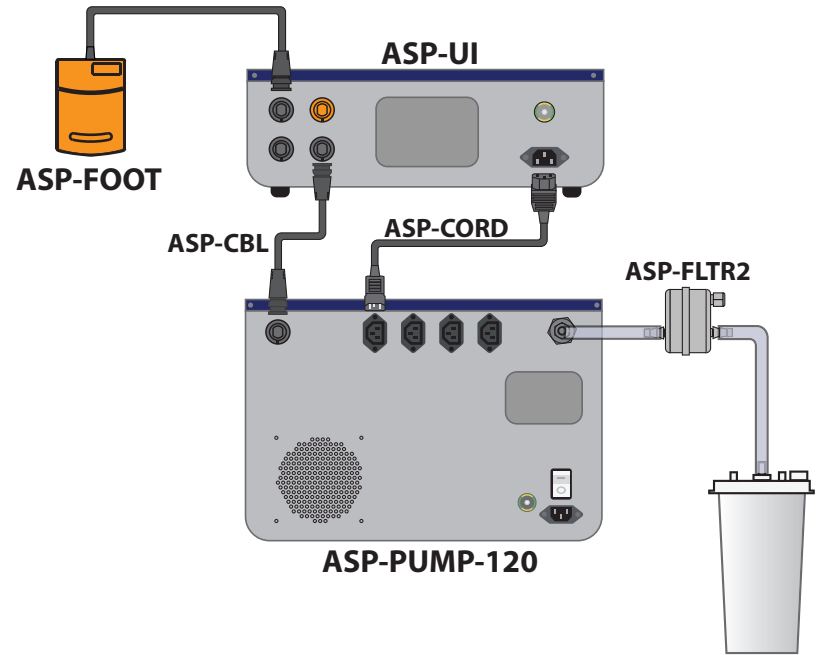

7020 + INF

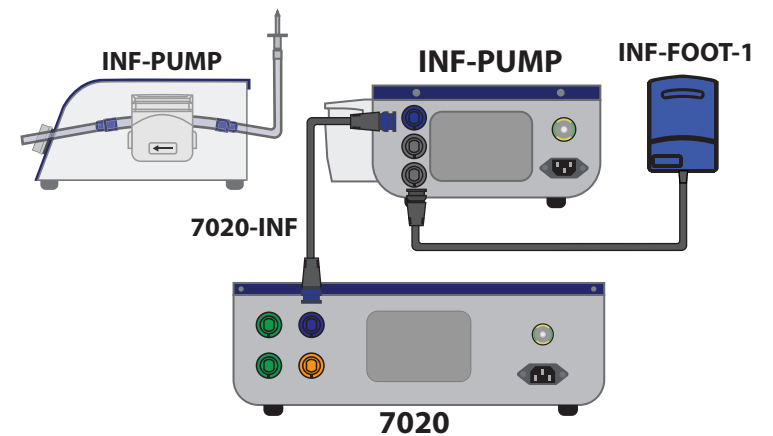


See IM-ASP-PUMP, IM-INF-PUMP,
and IM-7020 at microaire.eifu.com

MICROAIRE®

INF-PUMP

Direct control with INF-PUMP

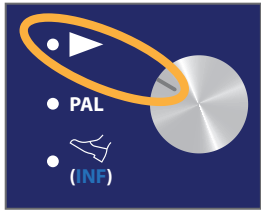


Activate Pump




Control set by 7020 console


(Activation with PAL handpiece or multi-pedal)

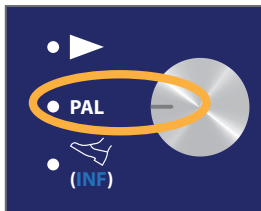

Control with INF-FOOT-1 pedal

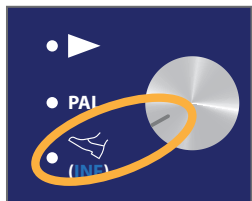

Activate Pump

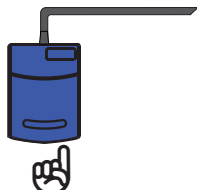

ASP-UI

Change Vacuum Pressure Units inHg ↔ mmHg



Reset Biofilter Timer

XXX.X h → 000.0 h




Change User-Preset Vacuum Pressure



7020

Switch Control Modes

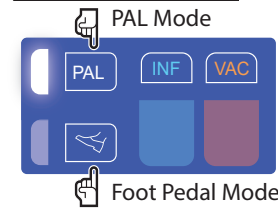

PAL Mode

Foot Pedal Mode

OR




7020-PEDAL

7020 + INF/ASP

Foot Pedal Mode

INF-PUMP:

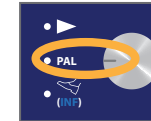

Activate Pump 





Activate INF Pump
Activate VAC Pump



PAL Mode

INF-PUMP:



Select Pump  



OR



Activate Pump 

