



PALSYNERGY®

More capabilities. More control.



Aspiration Tubing Connections

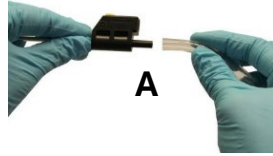


PAL-1200 Aspiration Tubing:
Connect small end to cannula (A) and large end to canister patient port (B) or fat processing device.

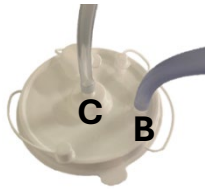

ASP-FLTR2 Biofilter & Tubing:
Connect one end to canister vacuum port (C) and other end to vacuum inlet on back of ASP Pump (D), with arrow on biofilter (E) pointing toward pump.

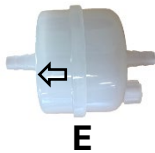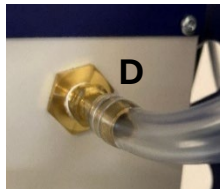

7020 Console Interface

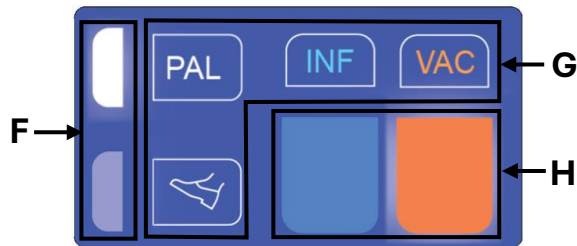


- F. Illuminated white block next to PAL or Foot Pedal indicates the selected control mode.
- G. Illuminated letters/symbol indicate that the corresponding device is properly connected and turned on.
- H. In PAL mode, illuminated blue or orange block indicates the Pump selected to sync with the Handpiece.

Infiltration and Aspiration Pump Control


Step 1: Choose Control Mode

Black Dome Button: Switch between Foot Pedal Mode and PAL Mode.

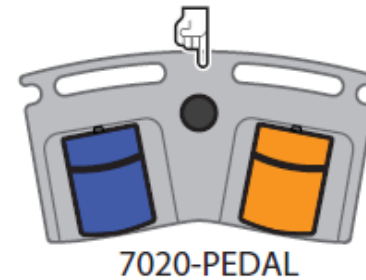


Step 2a: Foot Pedal Mode

Blue Pedal: Turn Infiltration Pump on and off.
Orange Pedal: Turn Vacuum Pump on and off.

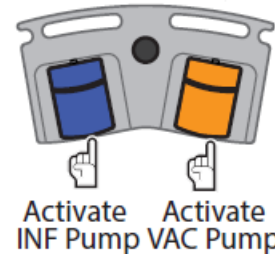




Step 2b: PAL Mode

Blue Pedal: Select Infiltration Pump, or
Orange Pedal: Select Vacuum Pump.
PAL Handpiece: Turn on selected pump when reciprocation is activated.

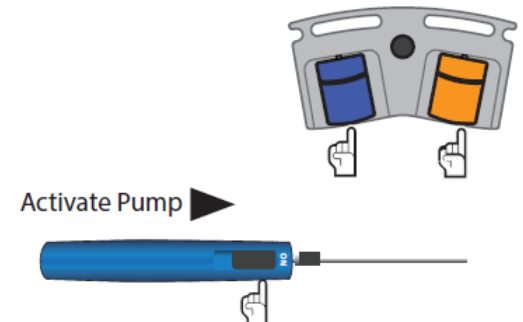

Troubleshooting

For complete troubleshooting guidance, see product IFUs at microaire.eifu.com.

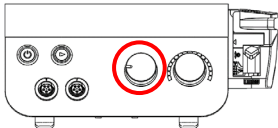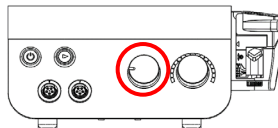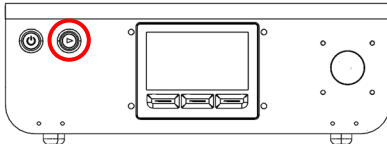Problem	Possible Cause	Solution
Infiltration/Aspiration Pump doesn't activate when Handpiece is activated.	Incorrect control mode selected on 7020.	Change to PAL Control Mode using the Foot Pedal black dome button or 7020 interface.
	Pump not selected for activation with Handpiece.	Use the Foot Switch (blue or orange pedal) or 7020 interface to select a pump to sync with the Handpiece.
	7020-INF/7020-ASP Cable loose or not plugged in.	Ensure the Cable is securely plugged into the 7020 Console and Pump.
	Wrong setting selected on Infiltration Pump. 	Change Infiltration Pump dial setting to PAL.
Infiltration/Aspiration Pump doesn't activate when Foot Pedal is pressed.	Incorrect control mode selected on 7020.	Change to Foot Pedal Control Mode using the Foot Pedal black dome button or 7020 interface.
	7020-INF/7020-ASP Cable loose or not plugged in.	Ensure the Cable is securely plugged into the 7020 Console and Pump.
	Foot Pedal cable loose or not plugged in.	Ensure the Foot Pedal cable is securely plugged into the 7020 Console.
	Wrong setting selected on Infiltration Pump. 	Change Infiltration Pump dial setting to PAL.
Aspiration Pump doesn't deactivate when Handpiece is deactivated or Foot Switch is pressed.	"Play" button on Pump Console is pressed. 	Press the "Play" button on the Pump Console to deactivate it.